

Quick Start Guide


Signia TV Sound with Signia MaX hearing aids

High quality stereo audio
streamed directly to your
Bluetooth-enabled Signia
hearing aids.



Powering TV Sound

1. Take the Micro USB cable supplied with your TV Sound
2. Connect the Micro USB connector to the TV Sound device
3. Connect the other end of the cable to the TV's USB port
4. Turn on the TV

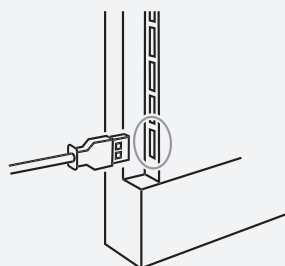


Note: If you do not have a USB port available on your television, plug the TV Sound to wall power with a USB power adapter (minimum 150 mA / 0.75 W).

Connecting the audio input


Via analog cable

1. Connect the analog audio cable to the back of the TV Sound (purple inlet)
2. Connect the two analog plugs to the corresponding sockets on the TV
3. Turn on the TV to start the audio

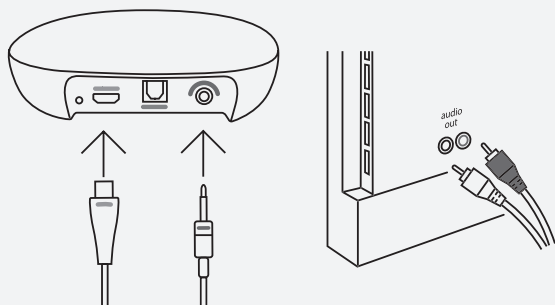

Via optical cable (recommended for best audio quality)

1. Connect the optical cable to the back of the TV Sound (blue inlet)
2. Connect the other end of the optical cable to the 'Optical OUT' socket on the TV
3. Turn on the TV to start the audio

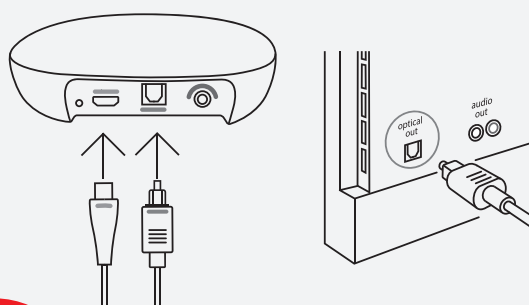

signia

Quick Start Guide

Signia TV Sound


Pairing your hearing aids to TV Sound

1. Power on the TV Sound
2. Switch the hearing aids off and on again by placing them in the charger and then removing them. This will put the hearing aids in pairing mode for 3 minutes
3. Place the hearing aids within 20 cm of the TV Sound
 - LED pulses blue while pairing
 - LED glows solid blue for 2 seconds and then fades out after successful pairing
 - LED glows solid green when there is an audio signal from the TV
 - LED pulses green if there is no audio signal from the TV

To pair to multiple sets of hearing aids, simply follow the same procedure again for each pair




Start streaming via the Signia One App

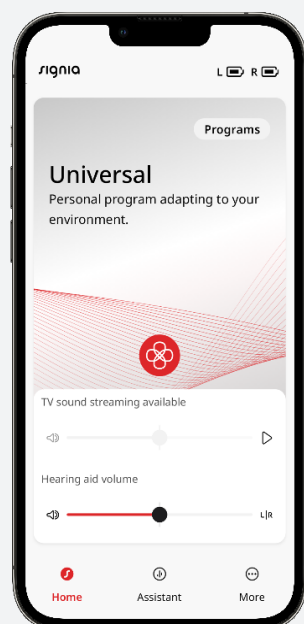

Step 1:

- Open the Signia One App
- Once hearing aids have been paired, the Play button for the TV Sound appears

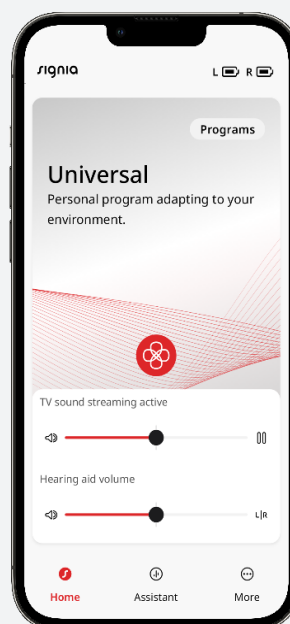

Step 2:

- Tap the Play button on the right side of the TV Sound volume slider to start streaming TV audio
- Tap Pause to stop streaming
- Adjust volume using the slider

Start streaming via the push button on your hearing aid

- Push the lower button on either the left or right hearing aid for about 3 seconds. Then release the button.
- You should now hear the TV audio in your hearing aids.

Note that your hearing care professional can change this functionality during the fitting session.